

CLEANING INSTRUCTIONS



Prior to starting the cleaning process please ensure the power is disconnected from the mains for the electric models



CLEANING

- Ensure the treatment couch is in a well ventilated and suitable area, prior to cleaning.
- For best results position the treatment couch in an upright position to allow for easier access of posterior areas.
- Wipe down the treatment couch with a dry soft white cotton cloth to remove excess dirt and dust.
- Clean the whole surface using liquid hand soap and water (1 part soap to 9 parts water).
- Do not use soap or cleaning solutions that contain alcohol, ketones, xylene, acetates or solvents (mineral/white spirits).
- If you choose to use a bleach solution, only use 0.5% concentration, which can be prepared as follows:
 - If the original concentration is 5.6% sodium hypochlorite, dilute in water with dilute 1 part of sodium hypochlorite per 10 parts of water.
 - If the original concentration is 8-10% sodium hypochlorite, dilute in water with dilute 1 part of sodium hypochlorite per 20 parts of water.
- Rinse and wipe with a clean damp soft white cotton cloth to get rid of any excess soap. Repeat as needed and then dry the surface.
- Ensure all corners and crevices are cleaned as these areas can store build up or micro-organisms.
- It is best to wipe in an 'end to end' motion.
- Although the surface is non-porous and water resistant, wipe the treatment couch dry prior to cleaning with disinfectant or other solutions to leave a streak free finish.
- Be very careful not to saturate the product. Saturation could lead to seepage into hinges and cause rust, which can result in the hinge ceasing and failing to work.
- Wash hands thoroughly after cleaning to avoid any possible irritation.
- For best results it is advised to clean the upholstery at least once a week to extend the lifespan. Once a month, you can use a soft-bristle brush for a deeper clean. Plastic or wire sponges, or hard-bristle brushes can damage the material permanently.

PLEASE NOTE

- For all sorts of stains clean within 24 hours to avoid permanent stains in the surface. After this time has elapsed, effectiveness drops significantly.
- It is advised to allow between 10 -15 minutes to thoroughly clean the TitanMed treatment couch.

Issue No. 2	Date effected: 01/08/21	Approved By: NB	Page: 1 of 1
-------------	-------------------------	-----------------	--------------

